IT	Adesivo cutaneo con puntale applicatore	1
EN	Skin adhesive with applicator tip.....	7
PT	Adesivo cutâneo com ponteira aplicadora.....	13
ES	Adhesivo cutâneo con punta aplicadora.....	19
FR	Colle cutanée avec embout applicateur.....	25
DE	Gewebekleber mit Applikatorspitze.....	31
PL	Dwuskładnikowy klej do skóry z końcówką aplikatora.....	37
CS	Lepidlo na pokožku s aplikačním hrotem	43
SK	Lepidlo na pokožku s aplikačným hrotem.....	49
SL	Kirurško lepilo za tkivo s konico za nanos.....	55
DA	Hudlim med applikatorspids.....	61
ET	Nahaliim koos aplikaatori otsikuga.....	67
FI	Iholiima levityskärjellä.....	73
NL	Huidlijm met borsteltje.....	79
SV	Hudlim med spetsapplikator	85
EL	Επιδερμική κόλλα δύο συστατικών με άκρο εφαρμογέα.....	91
SR	Двокомпонентни лепак за употребу на кожи са врхом за апликатор.....	97
HR	Dvokomponentno ljepljivo za kožu s vrhom aplikatora.....	103
RO	Adeziv pentru piele cu vârful aplicator.....	109
BG	Кожно лепило с връх за нанасяне.....	115

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OCA (2-octyl cyanoacrylate)

Features

Glubran® Tiss 2 skin adhesive is a class IIa surgical medical device that complies with current European regulations.

Glubran® Tiss 2 is a second-generation, two-component adhesive for skin use, supplied with a single-use applicator tip packaged in a blister. It is ready-to-use, sterile, cyanoacrylate-based.

It is a straw-coloured liquid that quickly polymerises on contact with tissues, creating a thin, impermeable, elastic film with high tensile strength, which ensures solid adhesion to tissues. The fluidity of the product and the characteristics of the applicator allow a controlled release of the adhesive even in areas that are difficult to treat. Once polymerized (60-90 sec.) it no longer has any adhesive power.

The adhesive after polymerisation provides an effective antimicrobial barrier against the bacteria responsible for the most common infections.

The product is compatible with most catheters used for central and peripheral venous access (CVC, PICC and Midline).

It is available in packages of 0.25 ml, 0.35 ml, 0.5 ml containing 10 blisters with sterile single-dose vial and applicator tip.

Indications for use

Treatment of skin tears and incisions not exceeding 7-8 cm in length.

It may be applied to all skin wounds caused by injury or surgical incisions.

It may be applied to catheter insertion sites for central and peripheral venous access in order to reduce their displacement, provide protection from bacterial infection through a barrier effect and induce haemostasis even in patients with congenital and non-congenital clotting disorders.

Indicated for paediatric use.

Indicated for the closure of surgical wounds during prosthodontic, maxillofacial and oral surgical procedures.

It may be applied to oral mucosal wounds caused by injuries or surgical incisions.

Modes of application

The application of Glubran® Tiss 2 is quick and easy:

1. Open the blister and remove the vial and tip.
2. Drain the product at the bottom of the vial by holding it upright with the cap facing upwards.
3. Remove the cap of the vial by turning it.
4. Insert the applicator tip onto the neck of the vial, making sure it is properly secured before applying the adhesive.
5. Turn the vial upside down so that the adhesive flows out at its end. Exert slight pressure on the body of the vial to soak the tip with adhesive. Take care not to press too hard to avoid spillage.
6. Making sure that the wound edges are close to each other, apply a thin layer of adhesive to the wound. An excessive amount of product delays polymerisation time, reduces the tensile strength of the adhesive at the point of application and induces improper wound healing.

The wound edges should be held together for about 1 minute. Once polymerisation has taken place, no further correction is possible.

The adhesive will detach spontaneously about 5-8 days after application.

Contraindications

The adhesive must never be applied inside wound flaps, and may only be applied to the skin once the previously cleaned and dried wound margins have been brought together.

Do not apply to infected, chronic or gangrenous wounds or those resulting from pressure sores.

Do not use the adhesive in subjects with hypersensitivity to the product.

Do not use on inflamed and/or oedematous tissue.

Do not use in the presence of pus and/or purulent exudates such as those caused by infectious processes.

Do not use the product in case of hypersensitive patients and in pregnant women.

Precautions

Before using the product, check the state of fluidity of the adhesive inside the vial and its transparency. If the product is not very fluid and/or cloudy, it must not be used.

The product must always be applied in minimum quantities. A second layer of adhesive may only be applied on top of the first after the latter has already polymerized.

Any excess product can be removed with a dry pad within the first 5-6 seconds after application.

Do not allow the adhesive to come into contact with the eyes. In case of accidental contact, wash immediately with water. If the product has polymerized, it will detach spontaneously after about 2-3 days.

Should the adhesive come into contact with surgical instruments, it can be removed with acetone.

In the days following application, avoid prolonged contact with water and detergents. Do not rub the treated surface and dry by gently wiping with a soft cloth.

Warnings

- ⚠ The adhesive must be used by physicians with expertise in the use of the product. The user is responsible for the correct use of the product.
- ⚠ Glubran® Tiss 2 is suitable for the treatment of recent, cleansed wounds, the flaps of which can be easily brought together. In case of tension at the margins, subcutaneous suturing is required.
- ⚠ Read all the information included in this instruction sheet carefully before use.
- ⚠ Do not use the adhesive if it has a viscous and/or cloudy appearance.
- ⚠ Do not use if the blister or single-dose is open or perforated. 
- ⚠ The product is single-dose. 
- ⚠ The device is sterile and cannot be reused and resterilised after first opening. Reuse involves risks of infection for the patient due to loss of sterility of the contents and impairment of the product's bonding capacity and functionality. 
- ⚠ The product is ready for use.
- ⚠ Unused leftover adhesive must be discarded.
- ⚠ In rare cases, modest, temporary inflammatory reactions may occur after application. The exothermic reaction that occurs during polymerisation, although contained within 50 °C, may sometimes cause a slight sensation of heat in some particularly sensitive patients (e.g. children or the elderly) at the point of application of the adhesive.
- ⚠ The product is not recommended for use to secure 100% silicone catheters.

Storage

The product must be stored between +2 °C and +8 °C. If it is necessary to store the product outside this temperature range, it may be kept for a maximum of 5 days at 22.5 °C ± 2.5 °C.

Validity

Properly stored, the product is valid for two years from the date of production. The product must not be used beyond its expiry date.

Sterility

The product is sterile. 

Disposal

Once used, dispose of the device according to local procedures and guidelines.

Contacts

Any serious incident that occurs, related to the device, should be reported to the manufacturer GEM S.r.l. at info@gemitaly.it and to the competent authority of the member state where the user and/or patient is established. In this case, a detailed description of the problem occurred is required, as well as the variable data of the medical device involved (sales code, batch number, UDI code).

Packaging and product codes

See table

REF	PACKAGE	FORMAT	GMDN
G-NBOC2	10 single-dose with tip/box	0.50 ml	34164
G-NBOC2-35	10 single-dose with tip/box	0.35 ml	34164
G-NBOC2-25	10 single-dose with tip/box	0.25 ml	34164

Symbols used

	Manufacturer's data		Do not re-sterilise
	Medical Device, the item is a medical device		Do not use if the packaging is damaged
	CE mark and Notified Body identification number		Single-dose / Do not reuse
	Code/catalogue number		See the Instructions for Use
	Expiry date		Unique device identifier
	Batch code		Production date
	Sterilised using aseptic techniques		Keep dry
	Sterile double barrier system		Keep out of direct sunlight
	Upper and lower temperature limits to which the medical device can be safely exposed.		

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